
Next Level Tech & Revolutionary Design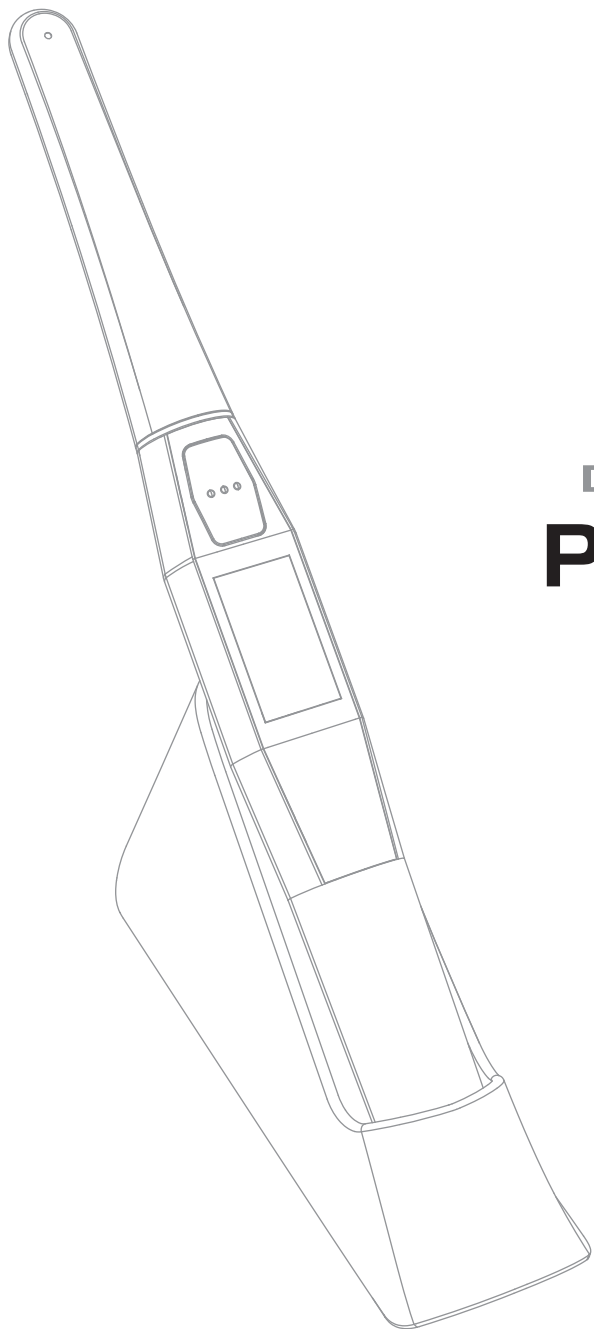


Dr's Light  
**PRIME**

User manual

ENG KOR

CE

Next Level Tech & Revolutionary Design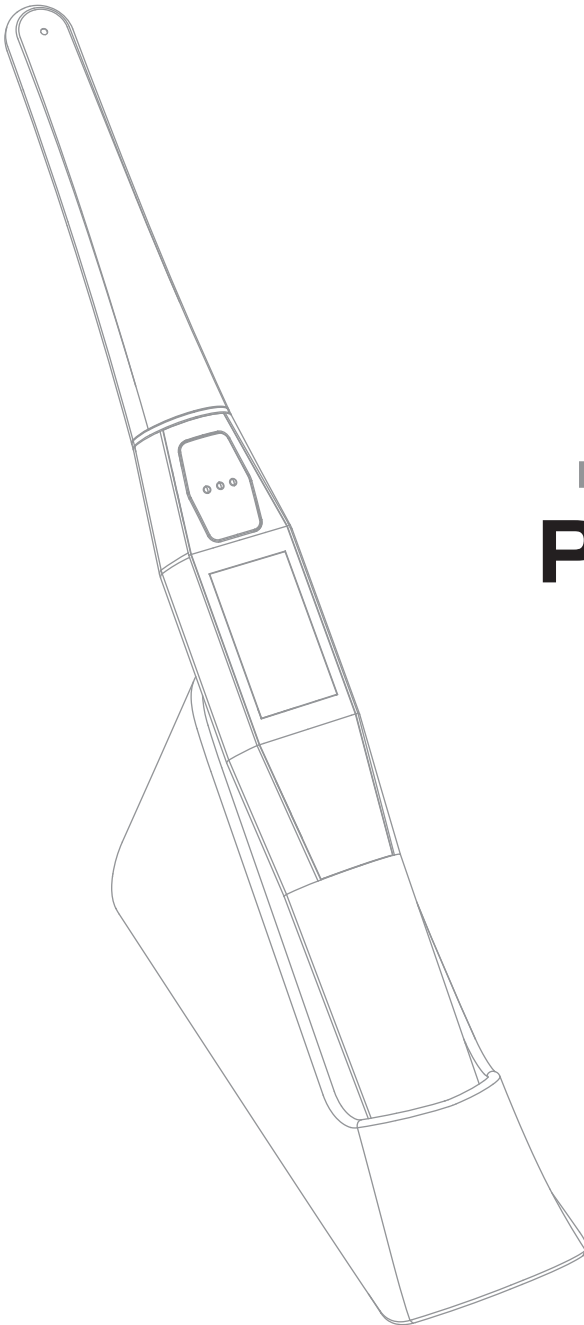

Dr's Light  
**PRIME**

User manual

ENG

CE

<b>1. Introduction</b>	04
- Features	
- Intended for use	
- Overview	
<b>2. Safety Information</b>	05
- Warning and Caution	
- Notes and References	
- Installation and Connection	
- Operation	
- Maintenance	
<b>3. List of components</b>	09
<b>4. Parts and functions</b>	10
<b>5. Installation</b>	11
<b>6. How to use</b>	13
<b>7. Maintenance</b>	17
<b>8. Problem-Solving</b>	18
<b>9. Specification</b>	19
<b>10. Accessories</b>	21
<b>11. Warranty</b>	21
<b>12. Warranty card</b>	22

# 1 Introduction

Thank you for purchasing the device.

The instructions described below have been designed to provide the necessary information for the successful operation, daily maintenance and safety for the use of this device.

## Features

---

- High Power Intensity
- Dual Wavelength (peak 405nm & 460nm)
- Slim Head Height
- Large Irradiation Area (12mm)
- Adjustable intensity power
- Highly Readable Color Touch Screen
- 360° twistable head
- Big Capacity of the Battery


## Intended for use

---

- Polymerization of light-cure catalyst containing dental materials with a wavelength spectrum of 400 ~ 490nm.

## Overview

---

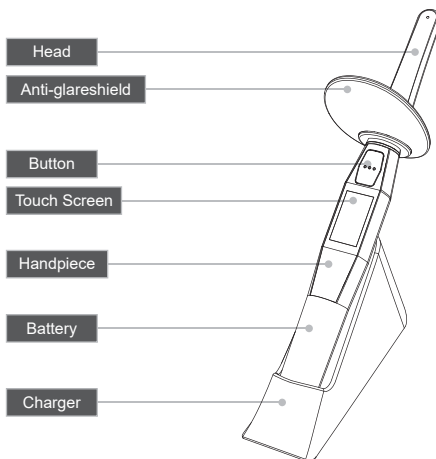## 2 Safety Information

It is important to understand the following information to use this product safely.

### Warning and caution

---

Warning and caution used in this manual are shown below.

**Warning**

Indicates a potentially hazardous situation which, if not avoided, may result in serious injury or death.

**Caution**

Indicates a potentially hazardous situation which, if not avoided, may result in minor/moderate injury or in damage of property.

### Notes and references

---

Notes and references used in this manual are shown below.

**Note**

Indicates that potential malfunctions may occur if the description is ignored.

**Reference**

Supplementary information to facilitate better operation.

## 2 Safety Information

### Installation and Connection

- The adapter should be disconnected from charger before connect the power cable to prevent an electric shock.
- Keep away from sources of ignition to avoid fire or explosion.
- Protect the handpiece, charger, and battery from any liquid to avoid electric shock or functional error.
- Place the charger on a flat and stable surface. Do not drop handpiece and charger to avoid internal defect that may cause overheating or fire.
- Do not place the electric cord near a gas burner or sources of heat. If the cord catch on fire, replace to new one to avoid short-circuit.

- Do not use or leave the unit in areas of high temperature or exposed to direct sunlight. Internal circuit may cause overheat or fire.
- Securely connect the power plug to the outlet with dust-free.
- Use this device in well ventilated areas. Overheating may cause fire.
- Do not touch the terminal areas of charger or battery with fingers or conductive objects such as metal wire or safety-pin. It may electric shock or fire.
- Do not use other charger, battery, or adapter. It may cause fire or device malfunction.

## 2 Safety Information

### Operation

#### Warning

- Do not plug the power cord with wet hands. This may result in an electric shock.
- In case of fluid leak from battery or partial discoloration / deformation of the handpiece, immediately stop use.
- Place the battery on correct direction when charging.  
Wrong directed placing may cause malfunction, explosion, or fire.
- In case partial discoloration of the charger, immediately disconnected the power plug and repair the charger.
- In case battery fluid contact the eye, immediately flush with copious amount of clean water and get medical attention.
- Do not use the device on patients with a history of cataract, retinal damage or allergic reaction to light.
- Do not use this device on a person with a pacemaker as it may cause malfunction of pacemaker.
- Avoid exposing the light on intraoral soft tissue as it may affect the soft tissue.
- Use disposable cover when use the device for patient to avoid cross-inspection.

#### Caution

- Disconnect the power cord by grasping the power plug. This may cause injuries or burns.
- Avoid damage or use of unnecessary force on the power cord. This may cause an electric shock or fire
- Should battery fluid contact the skin or clothes, immediately wash it off with copious amount water to avoid skin damage.
- Do not use this product for any purposes other than dental use.

## 2 Safety Information

### Maintenance

---

#### Warning

- Use only the battery supplied to avoid fluid leak or explosion.
- Never disassemble or overhaul the product.  
Unusual performance may cause injuries, electric shock, or fire.
- Ensure that charger and battery are always separated from the handpiece, and disconnect adapter cable from electric source prior to cleaning.


#### Caution

- Careful to avoid missing accessories.
- User must be in charge of operation, maintenance and check.


### 3 List of components


Please check the contents described below when removing the contents from the original packaging.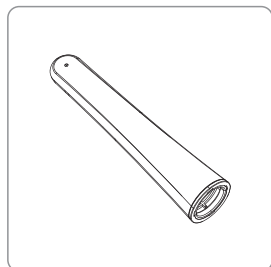


Head  
(Part No. DPM1101)

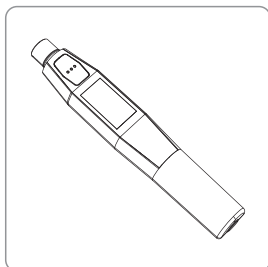
Handpiece  
(Part No. DPM1001)

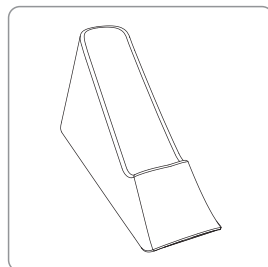
Charger  
(Part No. DPM2001)

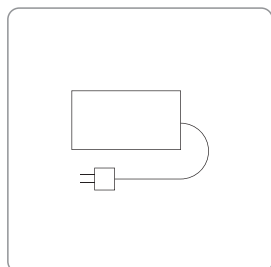
Adapter  
(Part No. AC0105)

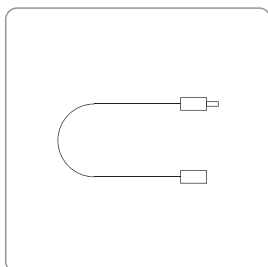Power Cable  
(Part No. AC0111)

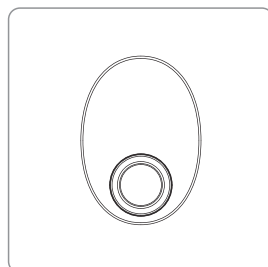
Anti-glare shield  
(Part No. AC0304)

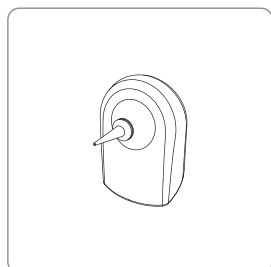M.A  
(Multi-purposed Attachment)  
(Part No. AC0303)


Disposable plastic cover  
(35 x 100mm / narrow)  
(Approx.100pcs)  
(Part No. AC0103)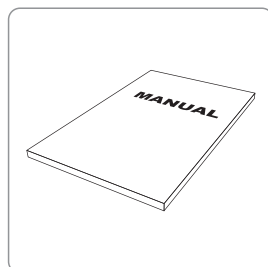

User Manual  
(Part No. DP080801)

## 4 Parts and functions
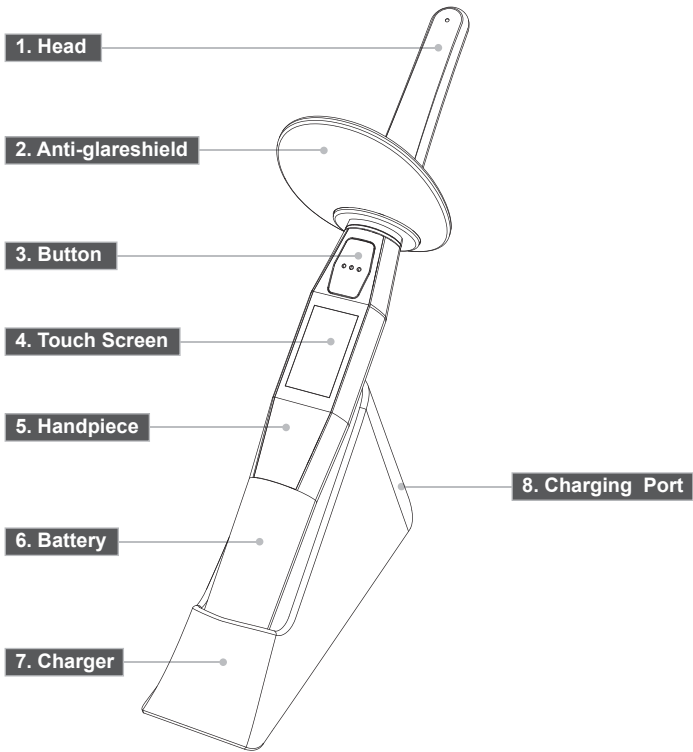
### 1. Head

- 360° twistable head
- Separable

### 2. Anti-Glare Shield

### 3. Button

- on / off button
- Start lightening and stop

### 4. Touch Screen

- Showing the condition of a product
- Select the program action

### 5. Handpiece

### 6. Battery

### 7. Charger


### 8. Charger Port

- AC adapter connection

## 5 Installation

Refer to the information supplied on safety precautions during installation and connection.

### 1. Connection between adapter and charger

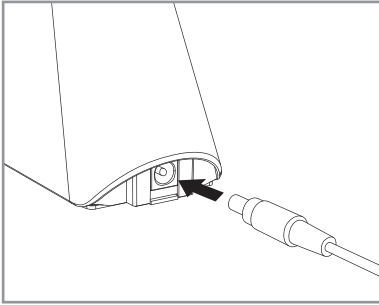After connecting the adapter and power cable, connect the power cable securely to the power outlet and connect the adapter connector to the back of the charger.

The adapter can be used from 100V to 240V.


**Caution**

Use only supplied adapter.


If the adapter is not supplied one, it may make charger malfunction.

### 2. Head Connection


Connect the head to the handpiece.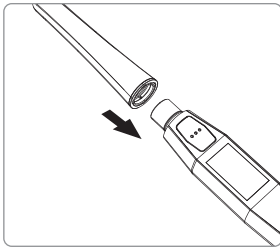

Please insert the head into the body as shown in the picture.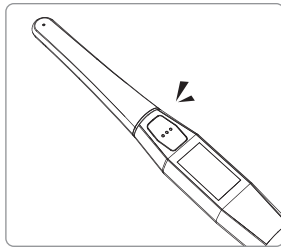

Insert the head toward the main body.

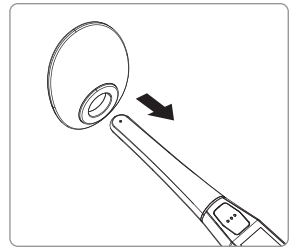Please insert the anti-glare shield as shown in the figure.


**Note**

Push the head and the main body to the end.






# 5 Installation

## 3. Charging

Please put the product on the charger.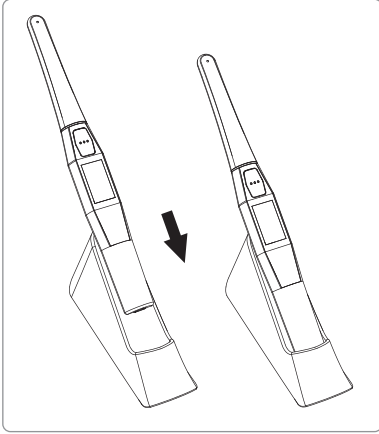

When charging starts, the touch screen will display the following screen.

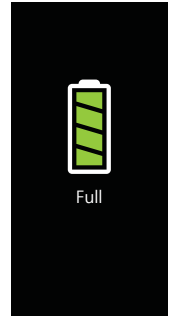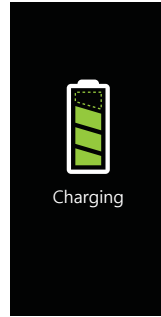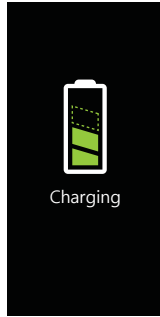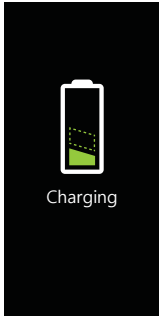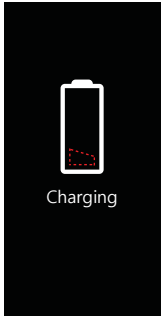battery discharge

fully charged

Display: current battery level  
Flickering: charging battery level gauge



When the battery is fully discharged, it takes 3-4 hours to fully charged.

## 6 How to use

### 1. Product Operation


When you press the irradiation button, the power is turned on and a beep sound is heard.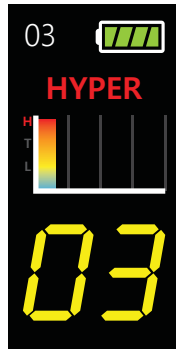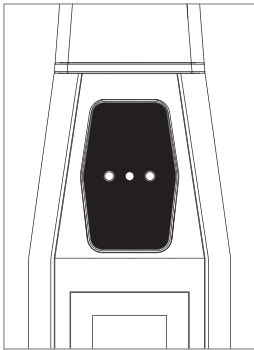

last mode of use

#### Touch screen configuration

##### ① Setting time

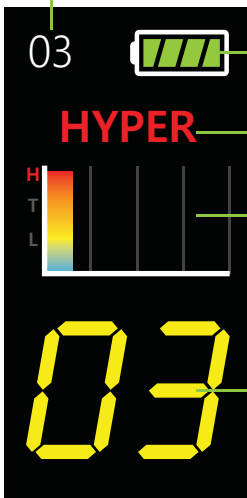##### ② Battery

Full charge

Need to charge

##### ③ Output mode

- HYPER(Original) > LOW > TRUBO > ORTHO > PULSE > SOFT.S cycle

##### ④ Output Graph

##### ⑤ Residual time.

- Yellow for less than 5 seconds.

- ① Setting Time: Display the currently set time
- ② Battery: indication of remaining battery capacity
- ③ Output Mode: Display current mode (last usage mode)
- ④ Output Graph: Visually display the current output level when the survey begins.
- ⑤ Residual time: Display the remaining hours of irradiation


1. The touch screen is a pressure type, it must be operated by applying a certain amount of force.
2. Mode change: You can change it by pushing the screen from top to bottom or from bottom to top.
3. Change the survey time: You can change it by pushing the screen from right to left or from left to right.
4. When you get the Low Bat sign, please charge or replace the battery.

## 6 How to use

### 2. Start Curing

Press the irradiation button to start the light irradiation in the selected mode.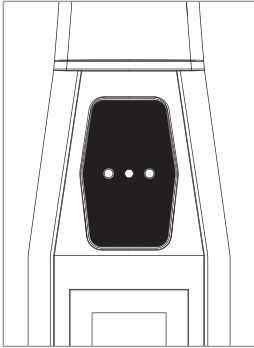


Reference

If you press the irradiation button during irradiation, the irradiation will stop.

### 3. Reset

In case the device seems work abnormally, try to device reset.

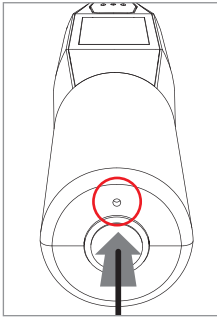Reset switch is under the bottom.

### 4. Automatic face off & Automatic power off

- If you do not press the product button for 30 seconds, the product screen automatically turns off to reduce power consumption. Please move the product to restart the screen.
- If you do not operate the product for an hour, the product will be powered off. Press the button to turn it on again.


Reference

During light irradiation, the automatic face-off and automatic power-off functions of the product are not applied.

## 6 How to use

### 5. Select the Curing mode


There are six built-in programs.

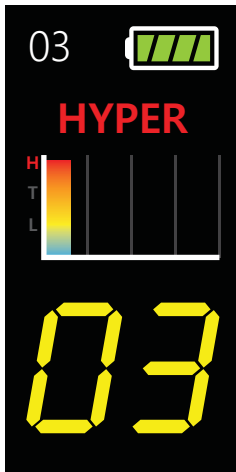① **HYPER** mode (3, 5sec)


② **LOW** mode (10, 20, 30sec)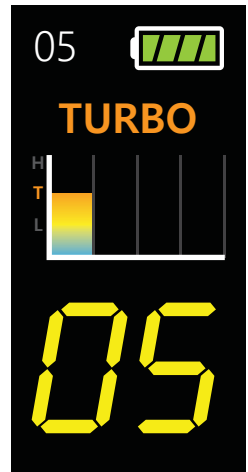


③ **TURBO** mode (5, 10sec)


Residual Frequency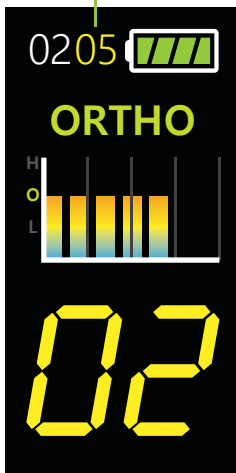

④ **ORTHO** mode  
(2sec x 5times)

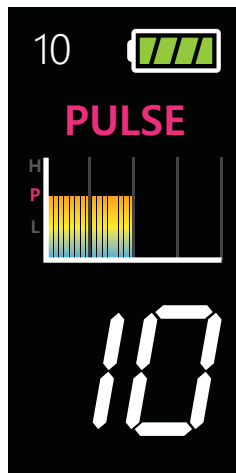⑤ **PULSE** mode (10, 20sec)

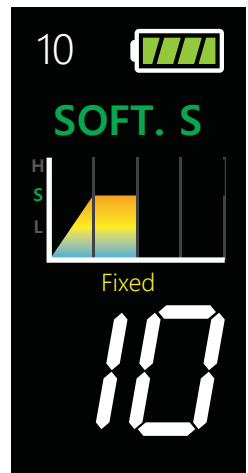⑥ **SOFT. S** mode (10sec fixed)


**Caution**


- When using Hyper mode, do not use it continuously for more than 5 seconds for fragile tissues such as gums. There is a risk of burns.

## 6 How to use


### 6. Replacement of the battery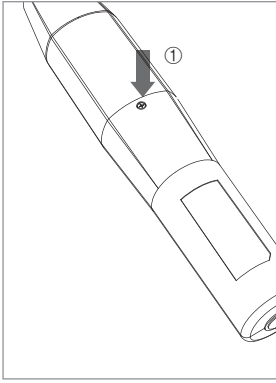


Loosen the screws on the rear battery cover of the product.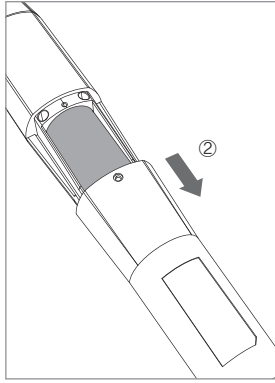


Please remove the battery cover.

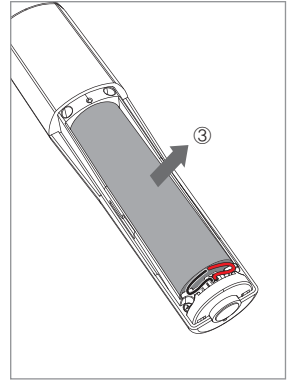
Please remove the battery.

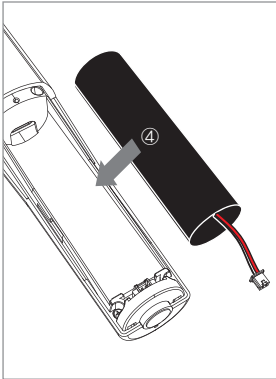Please insert the battery to be replaced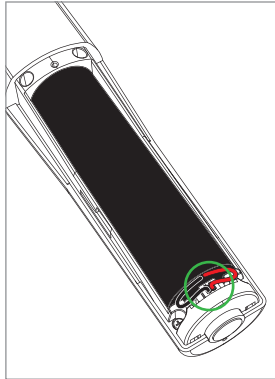

After replacing the battery, open the cable to the left and right as shown and close the battery cover.

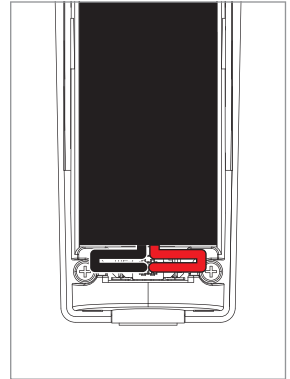**Caution**

- The cover cannot be fixed accurately without the battery fixing screw.
- Please be careful not to lose it.

## Cleaning and Disinfection

---

- Clean the device gently with clean, soft, lint-free cloth.
- Avoid using solvents which contain ethanol or acetone.
- Do not use sharp or pointed tools to clean as they may damage or scratch the plastic case.
- Do not spill any liquid into the device. It might make device malfunction.
- Take off the battery when the device cleaning to avoid electric shock.

## Maintenance and replacement of the battery

---

- High indoor temperatures can affect battery life and product performance.  
Please use it at room temperature.
- It is recommended that the battery be charged before use.
- If the number of fully charged batteries is reduced, please replace the battery.

## Maintenance

---

- In any case, do not arbitrarily dismantle the product.
- Please contact the supplier or manufacturer when repairing the product.
- If you are not using the product for your convenience, please charge it.

## Discard

---

Products cannot be disposed of as urban waste that is not classified within the European Union. Please follow the local regulations for disposal.

## 8 Problem-Solving

If the product does not work properly, please refer to the following.

Problem	causes and solutions
Button doesn't work	If the buttons are pressed with excess force, it will not work. In such case, press the button gently.
Display screen does not work	If the battery is completely discharged or connected incorrectly, display will not appear. Recharge the battery or reconnect the battery to reset the device.
Power off while in operation	If the operation button is pressed for a moment, power will be turned off. Press operation button again. Check the battery connection. If the battery is completely discharged, device can be turned off. Recharge the battery.
Not able to charge	If the power cord is not plugged in properly, it will not charge. Please check the code again. Please check the power terminal of the battery or charger for any foreign substances.
Light intensity is poor	The lens may be dirty, foreign substances or scratches on the lens may reduce the amount of light. Even if the product is normal, the user may feel that the amount of light irradiation of the product is low. Use the ingredients to see if they are polymerized well.
Device works abnormally	If the product is operating abnormally, please reset it. (see 14p)

\* If the symptoms do not correct after taking the above measures, please contact the supplier or manufacturer.

## 9 Specification

### ■ Equipment Class

Model:	CL-DP40
Trademark:	Dr's Light PRIME
Power supply:	3.7V Lithium ion battery
Protection from ingress of liquids:	IPX0 (ordinary equipment)

### ■ Operation, transport, and storage condition for the main unit

	Operation	Storage	Transport
Temperature (°C)	10 ~ 30	10 ~ 40	-10 ~ 50
Humidity (%)	10 ~ 80	20 ~ 80	20 ~ 80
Atmospheric pressure (hPa)	800 ~ 1360		

### ■ Additional Information

Handpiece (included curing light head)	Operation voltage	3.7V
	Wavelength range	400 ~ 490nm
	Measurement	Approx. 254 x 30 x 26 mm
Charger	Weight	Approx. 123.2 g
	Operation voltage	5V
	Measurement	Approx. 160 x 47 x 81 mm
Battery	Weight	Approx. 170.2 g
	Type	Lithium ion battery
	Nominal voltage	DC 3.7V
	Measurement	Approx. 68 x 19 x 19mm
Disposable cover	Weight	Approx. 46.6g
	Quantity	Approx. 100 pcs by weight
	Raw material	Polyethylene
Adapter	Input voltage	AC 100 ~ 240 V
	Output voltage	DC 5V 2A


\* This specification is subject to change without prior notice

# 9 Specification


## ■ Symbol

	This device conforms with the European Directive, 93 / 42 / EEC which includes the requirements for electromagnetic compatibility		
	This symbol is appixed to fulfill the requirements of EU Directive 2012/19/EU Article 11. This equipment cannot be disposed of as unsorted municipal waste within the European Union. Follow local regulations for disposal.		
SN	Serial number 00 00 00 0000 Model / Month / Year / Serial		Warning, Caution
	Type BF applied part		Manufacturer
	Authorized representative in the European Community		Check user manual.
	Prevent by wet.		This box is paper recyclable.
	Do not device on place where get direct sunshine.		Fragile item
	Transport temperature.		No hooks
	Reuse is not allowed		


## 10 Accessories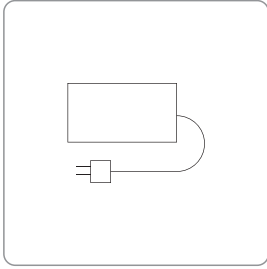


Adapter  
(Part No. AC0105)

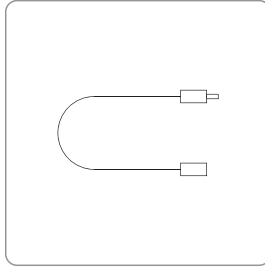Power cable  
(Part No. AC0111)

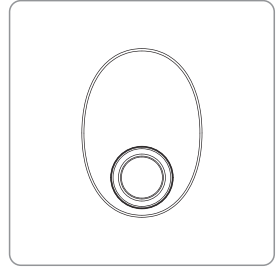
Anti-glare shield  
(Part No. AC0304)

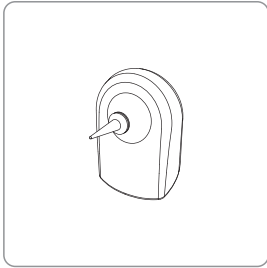MA  
(multi-purposed attachment)  
(Part No. AC0303)

Disposal plastic cover  
(35 x 100mm / narrow)  
(Approx.100pcs)  
(Part No. AC0103)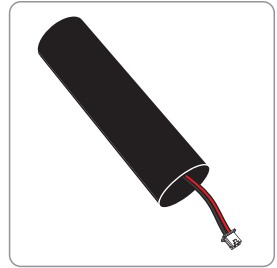


Battery cell  
(Part No. WHCD070501)


## 11 Warranty

This product is only shipped after strict inspection. Should the product incur any problems under normal use conditions within one year from the date of purchase, the repair will be free of charge. However, repair of consumables (Disposable cover, Anti-glare shield, Battery and etc.) are not covered by the warranty. For more information, refer 12. Warranty card attached.

### Checking of Serial Number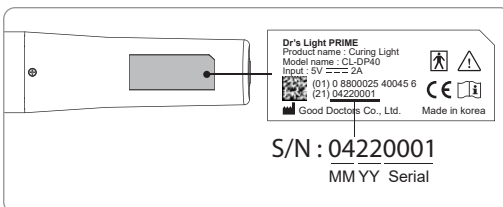

Please check the serial label on the back of the product.

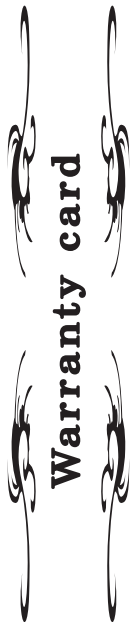## Warranty card

We guarantee the quality of the product as follows

**Name of product :**

**Name of model :**

**Customer information**

- Office name :
- Office phone :

**Distributor**

- Dealer name :

**Assurance period**

- Date of purchase : from \_\_\_\_\_ , 20\_\_\_\_
- Product is for 1 year except accessories.


**< Assurance details >**

1. If there is any defect of manufacturer or product has any fault itself within the assurance period, it will be replaced with the same product.
  - Malfunction and damage according to user's carelessness.
  - Malfunction and damage according to natural disaster such as fire, earthquake and flood etc.
  - Malfunction and damage according to user's intention or accident.
  - Malfunction and damage according to using incompatible product.
3. The manufacturer does not accept any liability resulting misuse.

**Good Doctors Co., Ltd.**  
 #208, B-dong, 283 Bupyong-daero, Bupyong-gu (Woolim Lions Valley, Cheongchoen-dong) Incheon 21315 REPUBLIC OF KOREA  
 Tel: +82 32 424 6325 Fax: +82 32 424 6326

Next Level Tech & Revolutionary Design

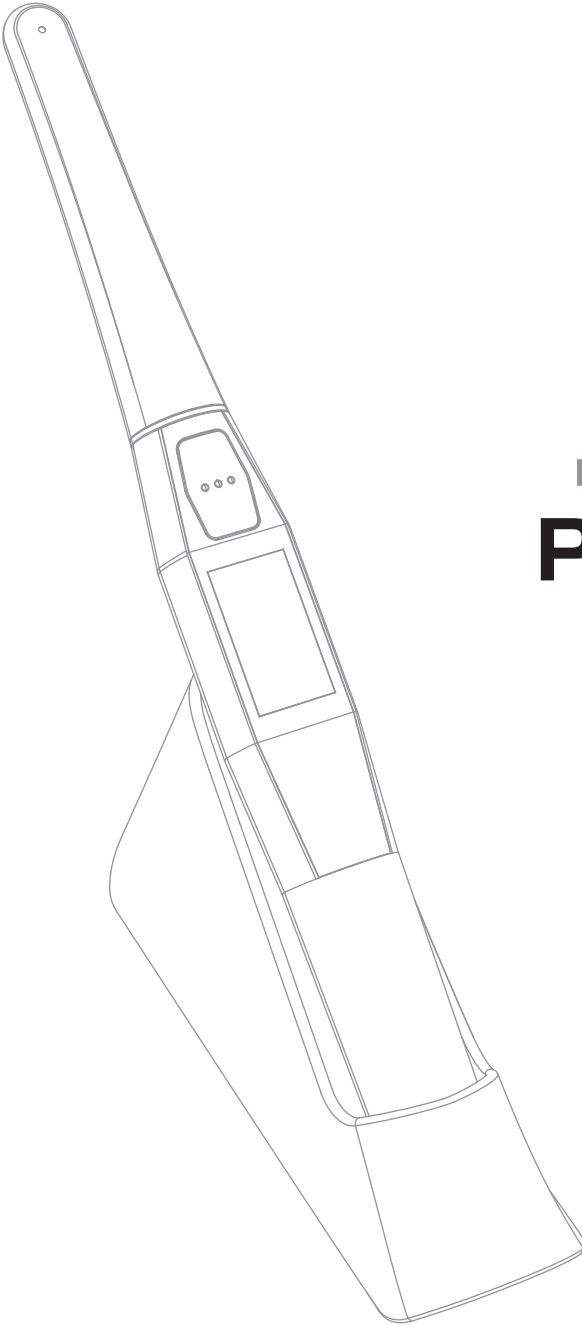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